

Process

Minimum 2-3 treatments
with 1 month interval

1 month

First session :
2~3 treatments

3~6 month

After first session, repeat cycle after
3~6 months

Hydro Lifting

Skin Rejuvenation

Specification

Contents : 2.7mL/syringe

Package : 1syringe/box

Ingredients : Low cross linked hyaluronic acid

Crosslinking rate : < 0.01

Molecule weight : 1200 kda

Certification

CE

93/42/EEC

Sterile Absorbable Hyaluronic
Acid Dermal Filler



Approved korean
medical device

조직수복용생체재료



Official Distributor in Indonesia

SK Interpharm

A New Vibe for Skincare & Aesthetic



Arielle

Skin Renewing Injectable
Long lasting Hydro Lifting



Official Distributor in Indonesia
SK Interpharm
A New Vibe for Skincare & Aesthetic

Aldeoz Cantina Lt. 07 / No. 45, Jl. Warung Jati Barat no. 39
RT/RW 010/004, Kalibata, Pancoran, Jakarta Selatan

✉ sales@skinterpharm.com ☎ 021-520-0046 ☎ +62811-9002-9533
{@} skinternational.id {#} www.skinterpharm.com

Next-Level Aesthetic Innovation

Arielle® Skinbooster is an advanced 5th-generation aesthetic treatment featuring hyaluronic acid (HA) for skin rejuvenation with CE certification. With stable HA architecture, it provides a firming effect, enhances the remodeling and repair process of the dermis, and reproduces hyaluronic acid and elastin for wrinkle improvement.



Efficacy





Use Guide

+ Face


Areas for 12~Point Injection, Inject 0.2cc per point
Short needle 34gauge, 1.8mm length, Face :
Vertical ~ 60 degree

High HA Minimally cross linked

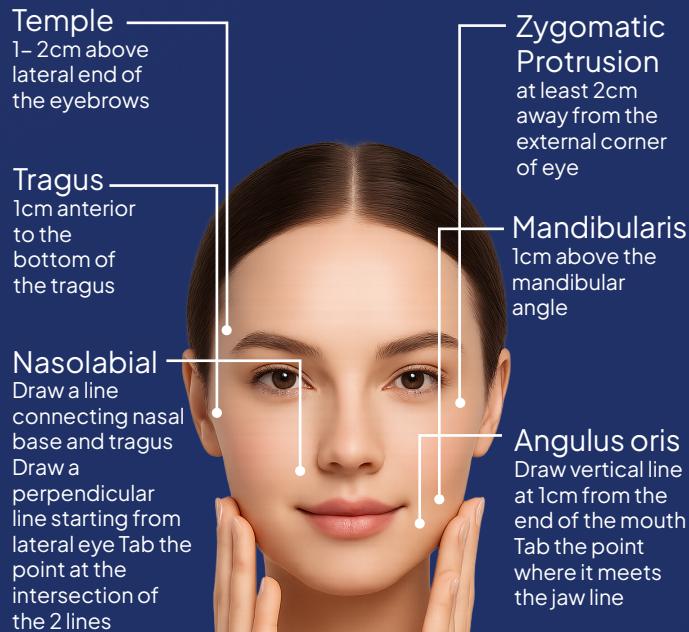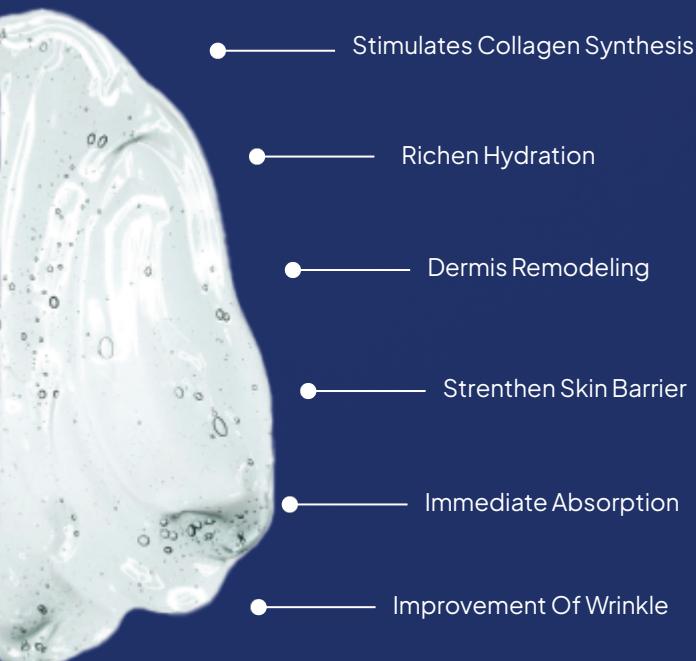

+ Neck

Areas for 10~15 Point Injection
Inject 0.1~0.2cc per point considering horizontal wrinkles over clavicle
Dermis
Needle depth 3mm
Inject avoiding blood vessel locations
Short needle 34gauge
1.8mm length
Neck : Vertical degree

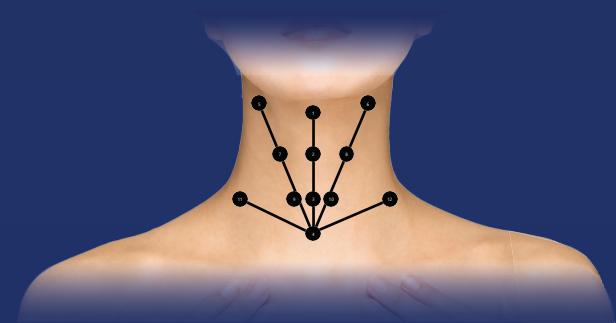

+ Hands

Areas for 10~12~Point Injection
Inject 0.1cc per point at distance of 1.5cm from the middle of the middle finger
Dermis
Needle depth 3mm
Inject avoiding blood vessel locations
Short needle 34gauge
1.8mm length
Hands : 60 ~ 30 degree

