Process

Minimum 2–3 treatments with 1 month interval

1month

First session:
2~3 treatments

3~6month

After first session, repeat cycle after 3~6 months

Hydro Lifting

Skin Rejuvenation

Spesification

Contents: 2.7mL/syringe

Package: lsyringe/box

Ingredients: Low cross linked hyaluronic acid

Crosslinking rate: < 0.01

Molecule weight: 1200 kda

Certification

CE 93/42/EEC

Sterile Absorbable Hyaluronic
Acid Dermal Filler



Approved korean medical device

조직수복용생체재료







Official Distributor in Indonesia **SK Interpharm**A New Vibe for Skincare & Aesthetic

Aldeoz Cantina Lt. 07/No. 45, Jl. Warung Jati Barat no. 39 RT/RW 010/004, Kalibata, Pancoran, Jakarta Selatan



Skin Renewing Injectable Long lasting Hydro Lifting




Next-Level Aesthetic Innovation

Arielle® Skinbooster is an advanced 5th-generation aesthetic treatment featuring hyaluronic acid (HA) for skin rejuvenation with CE certification. With stable HA architecture, it provides a firming effect, enhances the remodeling and repair process of the dermis, and reproduces hyaluronic acid and elastin for wrinkle improvement.



High HA Minimally cross linked

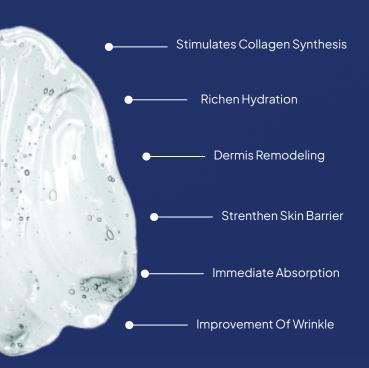

Efficacy



Use Guide


+ Face

Areas for 12~Point Injection, Inject 0.2cc per point Short needle 34gauge, 1.8mm length, Face : Vertical ~ 60 degree



+ Neck

Areas for 10~15 Point Injection
Inject 0.1~0.2cc per point considering horizontal
wrinkles over clavicle
Dermis
Needle depth 3mm
Inject avoiding blood vessel locations
Short needle 34gauge
1.8mm length
Neck: Vertical degree

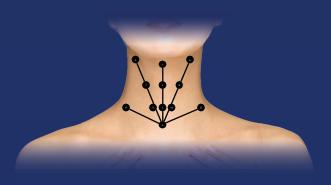

+ Hands

Areas for 10~12~Point Injection
Inject 0.1cc per point at distance of 1.5cm from the middle of the middle finger
Dermis
Needle depth 3mm
Inject avoiding blood vessel locations
Short needle 34gauge
1.8mm length
Hands: 60~30 degree

